

# Fortera<sup>®</sup>


Fortifying Osteoconductive Biocement



GFS-00082-01

## FORTERA<sup>®</sup> TIPS AND TRICKS

### PREP

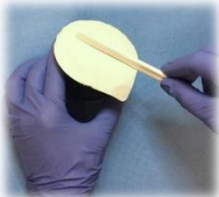

Ensure surgeon is ready

Tap unopened bowl on table to settle the powder before opening

Powder & Liquid = Premeasured

Slowly dispense all liquid into the bowl

Give an extra plunge on the liquid syringe to expel all liquid

CLOCK STARTS AFTER LIQUID IS ADDED

0:01


### MIXING

Spackle material against side of bowl (gently stirring is not enough)

Turn material over to reveal any dry particles

CONTINUE TO MIX FOR 1 MINUTE

Must achieve homogenous mixture before proceeding

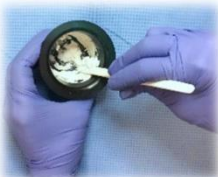


UNMIXED



MIXED

1:00

### HANDLING & DELIVERY SYSTEM

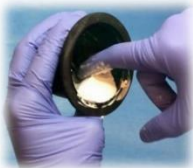

Use a gloved finger to gather the material to one side of the bowl and scoop it out - it may require multiple scoops

Immediately front load the delivery barrel syringe:  
Pull back piston slightly and use suction to draw in the material  
(recommended for all kit sizes)


Remove the cannula sleeve and discard

Attach the cap and cannula to the syringe barrel

Dispense the material into the defect

Mold and shape the material as needed

HANDS OFF AT 7 MINUTES FOR THE SETTING PERIOD

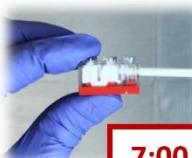

7:00

GFS-00142-01