


# Trabexus®

Trabecular Osteoinductive Biocement

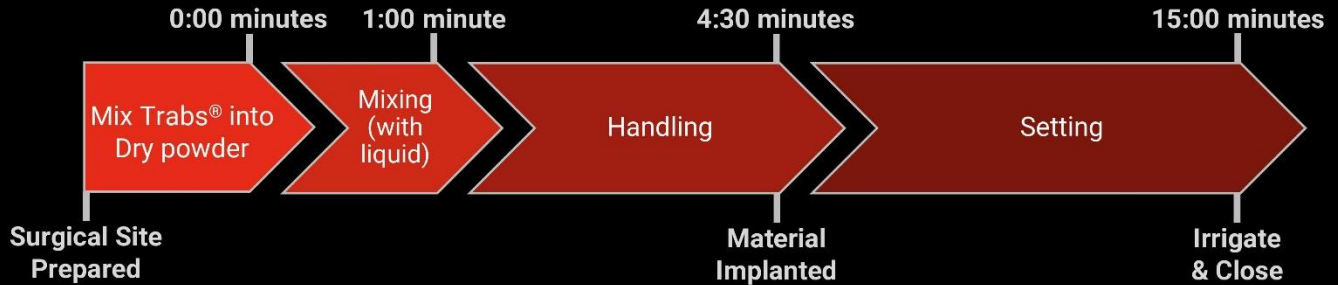

GFS-00081-01

## TRABEXUS® TIPS AND TRICKS

### PREP



Ensure surgeon is ready

Tap unopened bowl on table to settle the powder before opening

Powder, Trabs & Liquid = premeasured

Add all Trabs to powder & mix in bowl (+ Dry to Dry before Wetting)

Slowly dispense all liquid into the bowl

Give an extra plunge on the liquid syringe to expel all liquid

CLOCK STARTS AFTER LIQUID IS ADDED

### MIXING

Spackle material against side of bowl (gently stirring is not enough)

Turn material over to reveal any dry particles

CONTINUE TO MIX FOR 1 MINUTE

Must achieve homogenous mixture before proceeding



1:00

### HANDLING & DELIVERY SYSTEM (OPTIONAL)



Roll & knead material with hands to ensure a purely homogenous putty

Front load the delivery barrel syringe:

Pull back piston all the way and drop in a rolled cylinder (3cc or 5cc) OR Pull back piston slightly and use suction to draw in the material (10cc)

Attach the syringe cap and cannula & dispense the material

When extruding, apply an additional firm push on the barrel syringe piston

Plunger is needed for 3cc kit size

Mold and shape the material into the defect site

HANDS OFF AT 4½ MINUTES FOR THE SETTING PERIOD

4:30

GFS-00141-01